BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE
The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

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CONTRAINDICATIONS: Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND PRECAUTIONS
• If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
• We recommend that the first application be made under supervision of a health care professional.
• If worn directly on the skin, this type of shoe, free from injuries, moisture, oils, creams, gels or similar residues.
• The device can be worn with any type of shoe, although in some cases it will be necessary to call upon a shoemaker/repairer to adapt the shoes.
• Do not use over open wounds.
• Do not use this device on damaged and/or package has been opened.
NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

COMPONENTS
(A) One ankle cuff, with a posterior foam pad and 2 posterior supple stays.
(B) One elastic strap, with a metallic hook in the middle.
NB – When manipulating the device, always take care to point the metallic hook towards the ankle.
(C) An attachment system with a plastic plate and a rivet (3 sets delivered with the device) to adapt the device to all shoe types, where laces are too short (beyond metatarsal heads).
(D) The foot cuff for the device (not included).
(E) A mesh bag to transport or wash the brace.

APPLICATION INSTRUCTIONS
1) Open and apply the ankle cuff, as if applying a sock.
2) Position the ankle cuff around the ankle, above the malleoli (internal & external ankle bones). Tighten the cuff firmly and close it using the contact-closure. Push the cuff to verify it does not slide down. The 2 closure buckles should be centred at the back of the leg, at the top of the cuff.
3-A) – TO USE THE DEVICE WITH SHOES
– Choose an attachment point on the shoe (laces, buckle, eyelets) at the base of the first metatarsal (base of big toe).
– For shoes with laces: apply the shoe normally, tie the laces firmly, and attach the elastic strap's hook on several laces. If lateral correction needed, you can attach the hook into an eyelet on the shoe.
NB – Do not attach the hook on a single lace only, as the attachment might be too weak, and the effectiveness of the device diminished.
3-B) – TO ADAPT SHOES WITH NO ATTACHMENT POINT
– If there is no attachment point on the shoe (laces, buckles, eyelets), call upon a shoemaker/repairer to adapt the plastic plate and rivet correctly onto the shoe.
– Upon advice of a health care professional, choose the position of the attachment point on the shoe (laces, buckles, eyelets), call upon a shoemaker/repairer to adapt the plastic plate and rivet correctly onto the shoe.
3-C) – TO USE THE DEVICE BAREFOOT
– Wrap and close the foot cuff around the forefoot (at the base of the metatarsals / base of big toe). The attachment point (textile buckle) should be centred above the forefoot, at the base of the metatarsals. If lateral correction needed, the attachment point (textile buckle) can be positioned on the base of the second metatarsal (base of the middle toe).
4) Open the closure buckles and pass each elastic strap through the corresponding posterior closure buckle. With the foot in dorsiflexion (pointing upwards), tighten the elastic straps according to level of tension required. Ensure closure buckles are securely closed. If the edges of the elastic straps are too long, they can be cut, leaving a 4 cm or 5 cm of extra length.
5) To remove the device, just detach the elastic strap's hook from its attachment point.
Avoid loosening the posterior buckles in order to keep the same level of tension for further use.
NB – If the hook tends to come undone while foot is in dorsal flexion, twist the hook and attach it in the opposite direction, facing upwards.

CARE
Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry in fire. Do not iron. The brace can be washed in the mesh bag. No need to remove the elastic strap from the ankle cuff.

COMPOSITION
Cuffs: Elastomer, elastomeric, polyester, spandex, polyurethane, rubber Hook and Buckles: Polyformaldehyde, steel

FOR SINGLE PATIENT USE ONLY.
CAUTION: THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS.

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

AVANT D'UTILISER CE DISPOSITIF, LIRE ATTENTIVEMENT LES INSTRUCTIONS SUIVANTES ET D'ÉTENDREMENT. LE USO CORRECTO ES FUNDAMENTAL PARA EL FUNCIONAMIENTO APROPIADO DE ESTE DISPOSITIVO.

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